USA

GB Rapid Exchange Therapeutic Infusion Catheter

- F** Cathéter de perfusion thérapeutique à échange rapide
- D** Therapeutischer Schnallaustausch-Infusionskatheter
- E** Catéter de infusión terapéutica de intercambio rápido
- I** Catetere Rapid Exchange per l'infusione di agenti terapeutici
- P** Cateter de infusão terapêutica de troca rápida
- NL** Therapeutische infusiekatheter voor snelle uitwisseling
- DK** Behandlingsinfusionskateter til hurtigt udskiftning
- S** Snabbutbytbar terapeutisk infusionskateter
- FIN** Nopeasti vaihdettava terapeutinen infuusiokatetri
- N** Terapeutisk infusjonskateter for hurtig utveksling
- GR** Καθετήρας θεραπευτικής έγχυσης ταχείας εναλλαγής
- PL** Terapeutyczny cewnik infuzyjny typu "rapid exchange"
- J** 簡単に交換可能な治療用注入カテーテル
- CN** 快速交换治疗输注导管

ClearWay™ RX

Rapid Exchange Therapeutic Infusion Catheter

Instructions For Use

ATRIUM MEDICAL CORPORATION
5 Wentworth Drive
Hudson, New Hampshire 03051 U.S.A.
☎ 603-880-1433 📠 603-880-6718

ATRIUM EUROPE B.V.
Rendementsweg 20 B
3641 SL Mijdrecht, The Netherlands
☎ +31-297-230-420 📠 +31-297-282-653

ATRIUM AUSTRALIA-PACIFIC RIM PTY. LTD.
Level 6, 579 Harris Street
Ultimo NSW 2007 Australia
☎ +61 2 8272 3100 📠 +61 2 8272 3199

©2010 Atrium and ClearWay are trademarks of Atrium Medical Corporation. Rev.2010/01

002790

www.atriummed.com

DESCRIPTION

The Atrium ClearWay™ RX is a therapeutic infusion catheter for localized delivery of therapeutic agents to the coronary or peripheral vasculature. This rapid exchange catheter is .014" guidewire compatible. The 24cm distal portion of the catheter is dual lumen with the outer lumen for inflation of the balloon and inner lumen as a guidewire channel. ClearWay™ RX releases fluid through the microporous PTFE material during continuous low-pressure injection (1-4 ATM). Radiopaque marker(s) are located within the balloon treatment zone to assist in catheter placement under fluoroscopy. This catheter is non-pyrogenic and is for single patient use only.

INDICATIONS

The ClearWay RX is indicated for localized infusion or irrigation of various diagnostic and therapeutic agents into the coronary and peripheral vasculature. It is not indicated for use in the neurovasculature.

CONTRAINDICATIONS:

Do not use this device for percutaneous transluminal angioplasty or stent deployment.

WARNINGS

1. The microporous PTFE material must be purged of air prior to use with the distal protective sheath in place.
2. To avoid air emboli, do not use air to inflate the microporous PTFE material.
3. To minimize the risk of vessel damage or balloon damage, do not exceed the maximum recommended inflation pressure provided on the package.
4. All agents used with this device must be used in accordance with the manufacturer's instructions for use.
5. Do not expose this catheter to organic solvents or ionizing radiation.
6. Remove the distal protective sheath prior to catheter insertion.

PRECAUTIONS

1. Only physicians trained in angiographic and percutaneous interventional techniques should use this device.
2. The enclosed instructions for use provide technical guidance only, and do not eliminate the need for formal training in the use of interventional devices.
3. Do not force catheter passage or withdrawal without first determining the cause of catheter resistance under fluoroscopy.
4. Do not use this device if the sterile package has become damaged or previously opened.
5. Do not use automatic or high pressure injection systems as they may exceed maximum recommended inflation pressures and damage the PTFE material, catheter shaft or both.
6. ClearWay RX is intended for single patient use only. Do not reuse, reprocess or re-sterilize. Reuse, reprocessing or re-sterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death.

POSSIBLE COMPLICATIONS




- Hematoma at the access point
- Acute myocardial infarction
- Spontaneous thrombus formation and occlusion of the artery, vein or graft
- Cerebral circulatory disorders
- Unknown side effects with regard to various irrigating solutions and concentrations
- Possible flow surface disruption
- Hypotension
- Ischemia
- Palpitations
- Vessel dissection
- Anastomotic rupture or injury
- Restenosis
- Hemorrhage or hematoma
- Unstable angina
- Arrhythmias
- Allergic reaction to contrast medium

- Infection
- Coronary artery spasm
- Embolism
- Death

DIRECTIONS FOR USE

A. BALLOON PREPARATION

1. Remove ClearWay RX catheter from protective hoop.
2. DO NOT remove the protective sheath and stylet.
3. Flush a 3-way stopcock with sterile saline and connect it to the ClearWay RX hub.
4. Connect one syringe with sterile saline to the side port and the syringe with prepared therapeutic agent solution to the other port.
5. A 10cc (ml) or larger syringe, is recommended for use with this catheter.
6. The ClearWay RX catheter must be primed and purged of air with sterile fluid prior to patient use with the distal protective sheath in place.
7. The distal protective sheath must be kept in place over the PTFE balloon material during all purging steps.
8. Flush the ClearWay RX catheter with a positive prep until fluid is observed coming out of the ends of the protective sheath. Continue to flush with 3-5cc (ml) of sterile saline.
9. Do not pull a vacuum on the syringe when purging the PTFE balloon material.
10. Prior to insertion, remove the distal protective sheath and stylet and visually inspect the PTFE balloon material for any damage. Do not discard protective sheath.

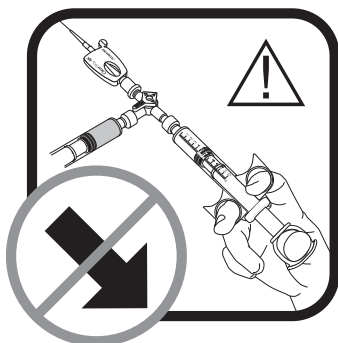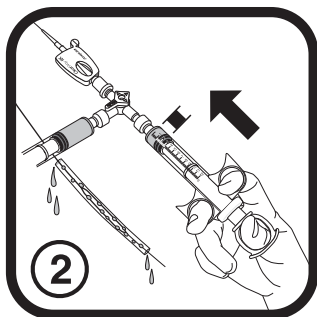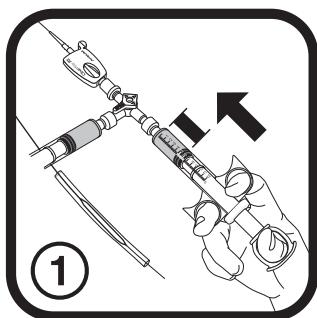B. CATHETER INSERTION

1. See corresponding package label for guidewire and guide catheter compatibility.
2. If necessary, the guidewire lumen can be irrigated with sterile saline solution prior to patient use.
3. Carefully introduce the ClearWay RX catheter tip onto the guidewire and insert into the guide catheter or introducer.
4. Once the ClearWay RX has passed through the hemostatic valve, back bleed the system to ensure that no air has entered the guide catheter.
5. Under fluoroscopy, advance the microporous balloon segment to the desired treatment location. Once placement is confirmed using the radiopaque markers, infusion can begin.
6. Turn the stopcock to open the path to the therapeutic agent infusion syringe.
7. Infuse prescribed amount of fluid using constant, steady pressure. Do not exceed the maximum recommended pressure found on the corresponding package label.
8. After infusion, flush system with 1-2cc (ml) of saline.
9. After flushing, immediately release syringe pressure by pulling back on the syringe barrel.
10. Pull a gentle negative pressure to deflate the balloon. Ensure that the device is deflated prior to withdrawal.

STORAGE

Store in a cool, dry place away from intense light.

SYMBOLS USED ON PRODUCT PACKAGING

REF CODE NUMBER

LOT LOT NUMBER

STERILE EO STERILE. STERILIZED BY ETHYLENE OXIDE

SEE PACKAGE INSERT

SINGLE USE ONLY

EXPIRATION DATE

Rx Only

PRESCRIPTION ONLY

BALLOON
PRESSURE

RX RAPID EXCHANGE

©2010 All Rights Reserved.

This device is covered under the following U.S. patent: 6395208B1

Atrium and ClearWay are trademarks of Atrium Medical Corporation.

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