Instructions for Use

- USA BAO Coated Polypropylene Mesh
- GB BAO beschichtetes Polypropylen-Netz
- F Treillis en polypropylène enduit d'huile bioabsorbable
- D BAO-beschichtetes Polypropylen-Netz
- E Malla de polipropileno revestida con aceite bioabsorbible
- I Maglia in propilene rivestita di BAO
- P Malha de rede de polipropileno revestida com óleo bioabsorvível
- NL Polypropyleen mesh met BAO-laag
- DK BAO-belagt polypropylenet
- S BAO-belagt polypropylenät
- FIN BAO-päällysteinen polypropyleeniverkko
- N BAO-belagt polypropylenett
- GR Επικαλυμμένο πλέγμα πολυπροπυλενίου BAO
- PL Polipropylenowa siatka powleczona olejem bioochłaniającym (BAO)
- J BAO コーティング済みポリプロピレンメッシュ
- CN BAO 涂层聚丙烯修补网

ATRIUM MEDICAL CORPORATION
5 Wentworth Drive
Hudson, New Hampshire 03051 U.S.A.
☎ 603-880-1433 📠 603-880-6718

ATRIUM EUROPE B.V.
Remdemsensweg 20 B
3641 SL Mijdrecht, The Netherlands
☎ +31-297-230-420 📠 +31-297-282-653

ATRIUM AUSTRALIA-PACIFIC RIM PTY. LTD.
Level 6, 579 Harris Street
Ultimo NSW 2007, Australia
☎ +61-2-8272-3100 📠 +61-2-8272-3199

©2010
Atrium and C-QUR Lite are trademarks of Atrium Medical Corporation.
Rev. 2010/04

ATRIUM MEDICAL CORPORATION
5 Wentworth Drive
Hudson, New Hampshire 03051 U.S.A.
☎ 603-880-1433 📠 603-880-6718

ATRIUM EUROPE B.V.
Remdemsensweg 20 B
3641 SL Mijdrecht, The Netherlands
☎ +31-297-230-420 📠 +31-297-282-653

ATRIUM MEDICAL CORPORATION
5 Wentworth Drive
Hudson, New Hampshire 03051 U.S.A.
☎ 603-880-1433 📠 603-880-6718

ATRIUM EUROPE B.V.
Remdemsensweg 20 B
3641 SL Mijdrecht, Pays-Bas
☎ +31-297-230-420 📠 +31-297-282-653

ATRIUM MEDICAL CORPORATION
5 Wentworth Drive
Hudson, New Hampshire 03051 U.S.A.
☎ 603-880-1433 📠 603-880-6718

ATRIUM EUROPE B.V.
Remdemsensweg 20 B
3641 SL Mijdrecht, Niederlande
☎ +31-297-230-420 📠 +31-297-282-653

ATRIUM MEDICAL CORPORATION
5 Wentworth Drive
Hudson, New Hampshire 03051 U.S.A.
☎ 603-880-1433 📠 603-880-6718

ATRIUM EUROPE B.V.
Remdemsensweg 20 B
3641 SL Mijdrecht, Países Bajos
☎ +31-297-230-420 📠 +31-297-282-653

ATRIUM MEDICAL CORPORATION
5 Wentworth Drive
Hudson, New Hampshire 03051 U.S.A.
☎ 603-880-1433 📠 603-880-6718

ATRIUM EUROPE B.V.
Remdemsensweg 20 B
3641 SL Mijdrecht, Olanda
☎ +31-297-230-420 📠 +31-297-282-653

ATRIUM MEDICAL CORPORATION
5 Wentworth Drive
Hudson, New Hampshire 03051 U.S.A.
☎ 603-880-1433 📠 603-880-6718

ATRIUM EUROPE B.V.
Remdemsensweg 20 B
3641 SL Mijdrecht, Holanda
☎ +31-297-230-420 📠 +31-297-282-653

ATRIUM MEDICAL CORPORATION
5 Wentworth Drive
Hudson, New Hampshire 03051 U.S.A.
☎ 603-880-1433 📠 603-880-6718

ATRIUM EUROPE B.V.
Remdemsensweg 20 B
3641 SL Mijdrecht, Nederland
☎ +31-297-230-420 📠 +31-297-282-653

Instructions For Use

Device Tracking Labels
The enclosed Device Tracking Labels should be attached to the dossiers of hospital records.

Description
Atrium C-QUR LITE™ Mesh is a sterile, knitted polypropylene monofilament mesh material for tissue reinforcement with a coating derived from a biological oil composed of fatty acids, lipids and glycerides (BAO - Bioabsorbable Oil Coating).

Indications For Use
Atrium C-QUR LITE Mesh is intended for use in hernia repair, chest wall reconstruction, traumatic or surgical wounds and other fascial surgical intervention procedures requiring reinforcement with a non-absorbable supportive material.

Contraindications
Atrium C-QUR LITE Mesh is contraindicated where tissue may be contaminated or infected and in infants, children or pregnancy where future growth may be compromised by its use.

Warnings

- Do not re-sterilize C-QUR LITE Mesh.
- Avoid direct contact with the viscera (intestines) to minimize the possibility of adhesions.
- Adequate mesh fixation is required to minimize post operative complications and recurrence. The fixation technique, method, and products used (including sutures, tacks, staples or other means) is left to the discretion of the surgeon to optimize clinical outcomes.

Precautions

- Handling of mesh should be with clean, sterile gloves and/or instruments.
- Careful attention to surgical mesh handling, suture, staple, or tackler fixation is required in the presence of nerves and vessels in the surgical field.

Adverse Reactions
Complications that may occur with the use of any surgical mesh include, but are not limited to, inflammation, infection or mechanical disruption of the tissue and/or mesh material, possible adhesions when placed in direct contact with the viscera (intestines).

Open Sterile Package
Peel open the package and remove the Atrium C-QUR LITE Mesh using sterile technique.

Handling & Operative Techniques
The Atrium C-QUR LITE Mesh must be removed from the handling sleeve using sterile technique prior to use.

Atrium C-QUR LITE Mesh products should be shaped, sized and anchored taking into consideration the patient's posture, weight and anatomical location. In order to minimize anchor means disruption, the C-QUR LITE Mesh should be sized and positioned so that it extends adequately beyond the margins of the defect.

Careful attention to suture/staple/tackler placement and spacing will help prevent excessive tension or disruption of the mesh material and/or underlying tissue. It is recommended that suture/staples/tacklers be placed a minimum of 5 mm from the edge of the mesh for best results.

Sterilization & Storage
Atrium C-QUR LITE Mesh is an ETO sterilized single use device; must not be re-sterilized. Store at 25°C (77°F) or less. Brief exposure up to 40°C (104°F) is acceptable.

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C-QUR LITE™ and Atrium are trademarks of Atrium Medical Corporation. ©2010 All Rights Reserved.