

Oasis™

DRY SUCTION WATER SEAL CHEST DRAIN



PRE-PACKAGED*
WATER



SET UP

- Step 1. Fill Water Seal to 2cm Line**
Add 45ml of sterile water or sterile saline via the blue suction port located on top of the drain. A pre-packaged water ampoule is provided on the back of each Oasis drain (*illustration 1*).
Twist top off the ampoule and insert tip into suction port (*illustration 2*). Squeeze contents into water seal until fluid reaches 2cm fill line.
- Step 2. Connect Patient Tube to Patient**
Connect chest drain to patient prior to initiating suction.
- Step 3. Connect Suction to Chest Drain**
Attach suction line to blue suction port on top of chest drain (*illustration 3*).
- Step 4. Turn Suction Source On**
Increase suction source vacuum to -80mmHg or higher. Suction regulator is preset to -20cmH₂O. Adjust as required.

100%
Latex-Free

100%
MRI Compatible



illustration 1

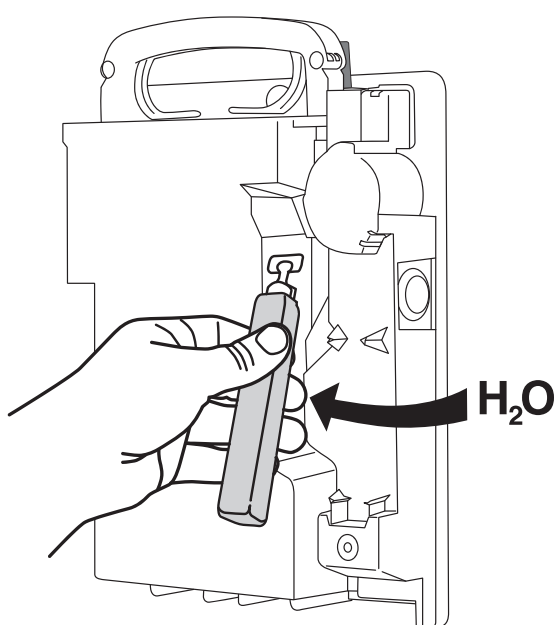


illustration 2


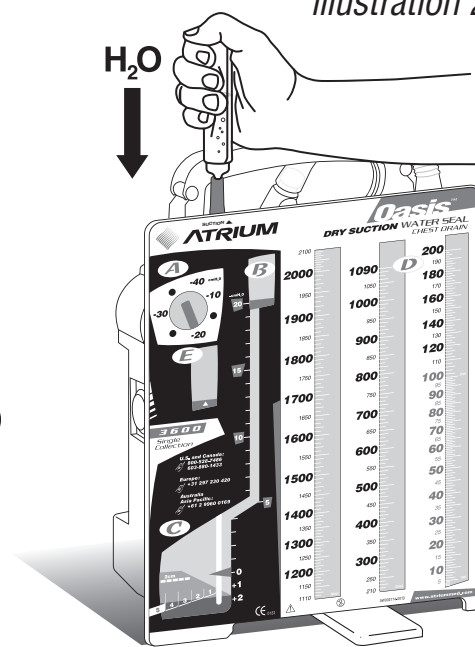


illustration 3



*Patented

Have a question or need help in a hurry?
Call Atrium toll free at 1-800-528-7486.

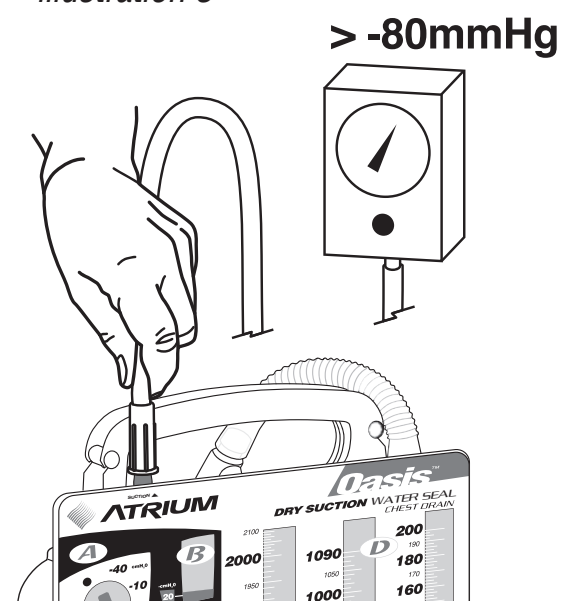

For additional education resources,
visit us at atriumuniversity.com.



ATRIUM